C H U | U V C
B R U G M A N N

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Manuel sommaire

CHU|UVC
BRUGMANN

[Information](#)

CHU|UVC BRUGMANN

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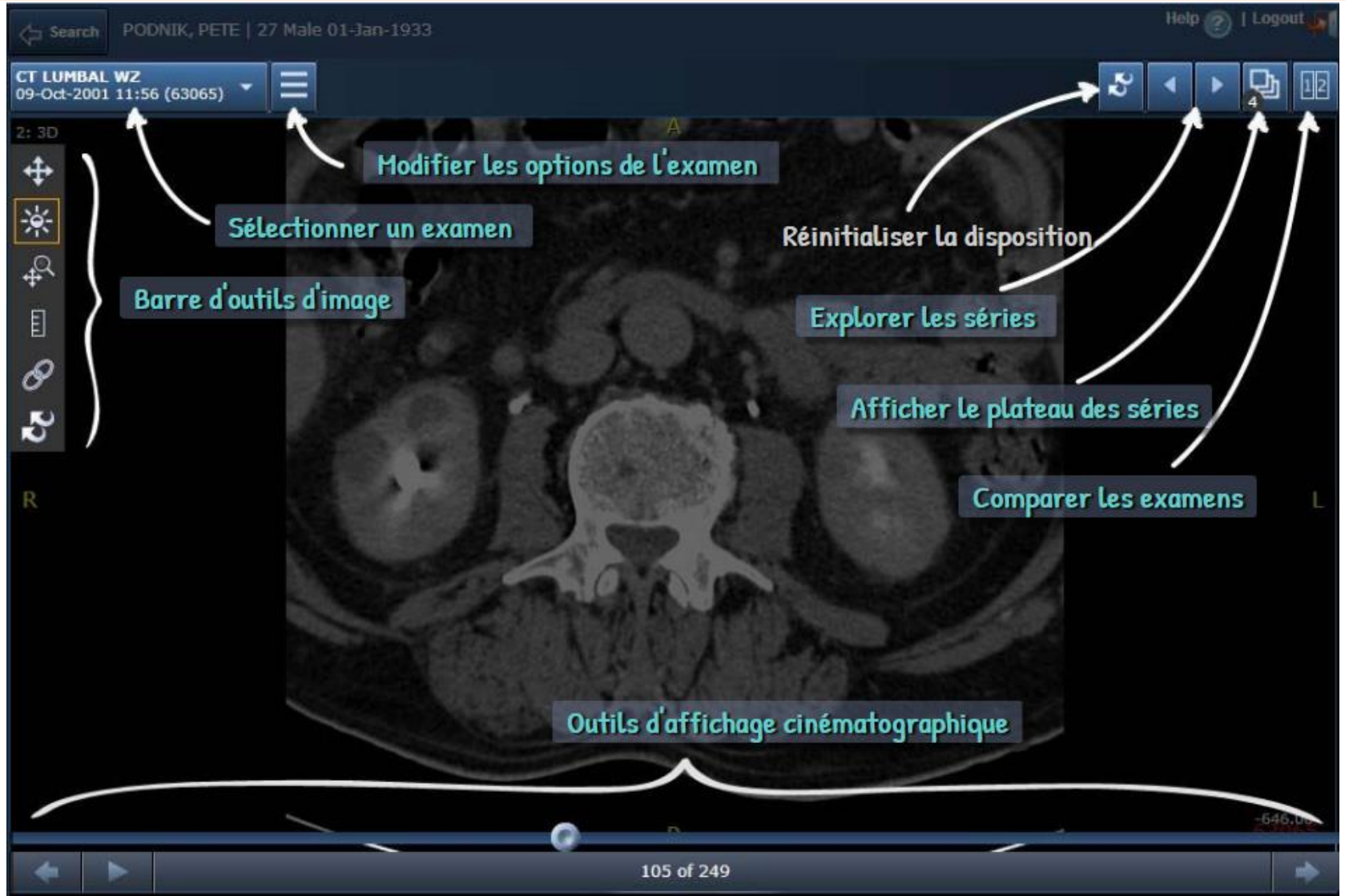
MENU PRINCIPAL

OPTIONS D'AFFICHAGE

The screenshot shows a medical software interface with a dark theme. At the top, there is a search bar and patient information: "PODNIK, PETE | 27 Male 01-Jan-1933". Below this, the study name "CT LUMBAL WZ" and date "09-Oct-2001 11:56 (63065)" are visible. A central menu is open, listing several options with checkboxes and radio buttons. The background shows a grayscale CT scan of a lumbar vertebra. On the left, there are navigation icons for 3D view, zoom, and other functions. At the bottom, there is a "Début" button and a page indicator "105 of 249".

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-646.00

Début

The image shows a cross-sectional CT scan of the lumbar spine. The interface includes a search bar at the top with the patient's name and date of birth. Below that, the study name and date are displayed. On the left, there are navigation icons for 3D view, zoom, and other functions. The main area shows the CT scan with a blue slider at the bottom for navigating through the series. The slider is currently at 105 of 249 images. Annotations in French explain the navigation controls: 'Reculer d'une image' (Previous image), 'Lecture' (Read), 'Faire glisser pour explorer les séries' (Drag to explore series), 'Nombre d'images' (Number of images), and 'Afficher l'image suivante' (Show next image). A 'Début' (Start) button is also visible on the scan image.

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
-6



i

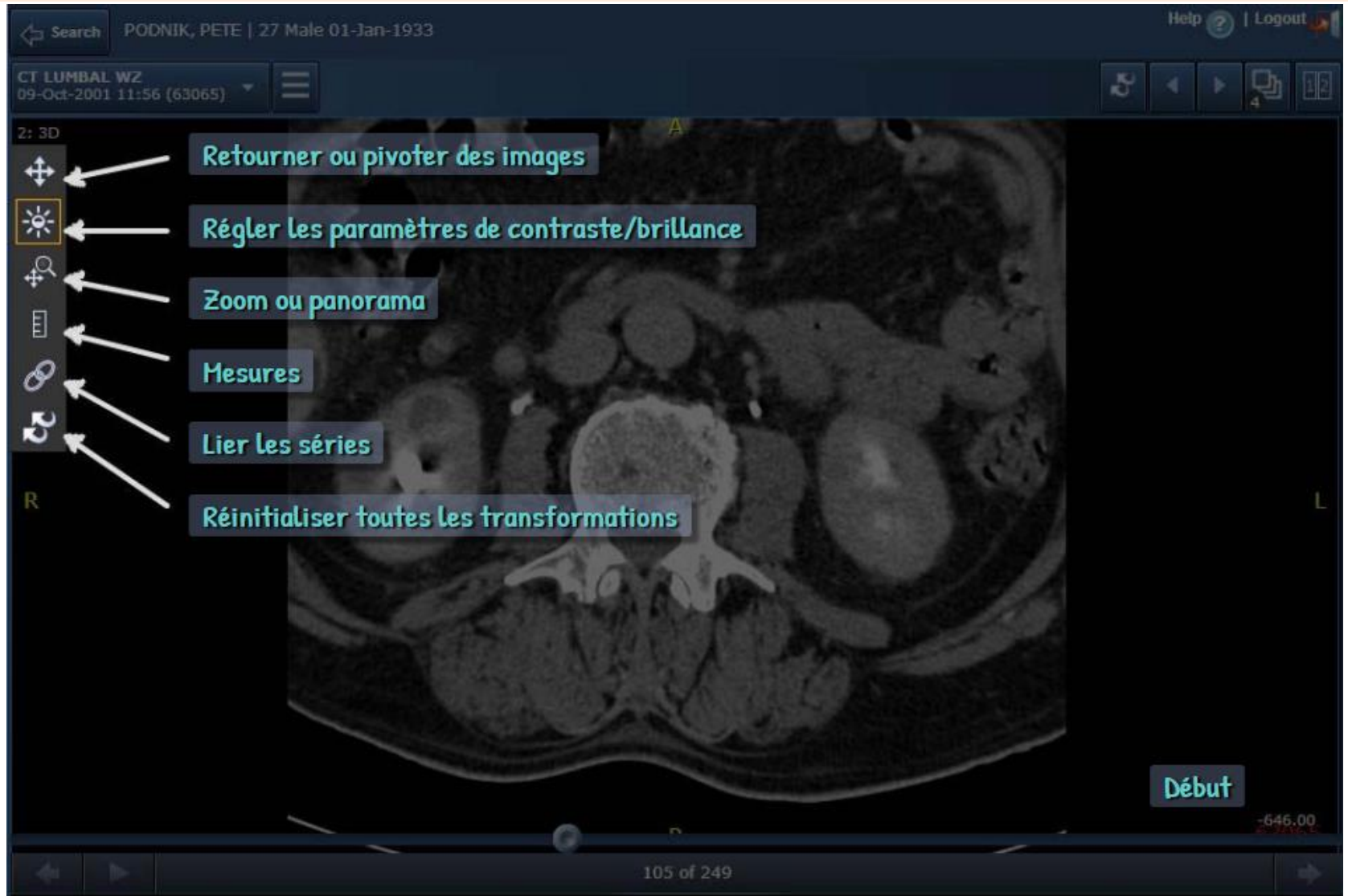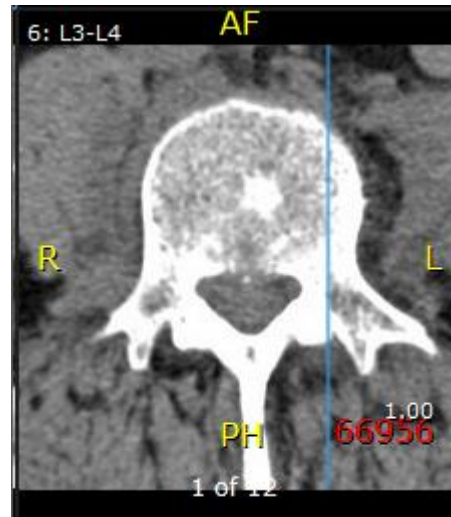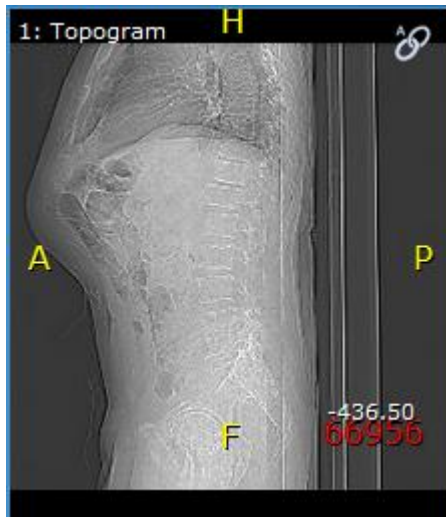The image shows a medical software interface in Cineview mode. The main window displays a CT scan of a lumbar vertebra. The interface includes a search bar at the top with the patient name 'PODNIK, PETE' and date '01-Jan-1933'. Below the search bar, the study name 'CT LUMBAL WZ' and date '09-Oct-2001 11:56 (63065)' are visible. The interface is divided into several sections: a left sidebar with navigation icons, a central image area, and a bottom control bar. The control bar contains a play/pause button, a 'Pause' button, and a timeline with a slider. Annotations with arrows point to various controls: 'Aller-retour' and 'Boucle' (loop) buttons, a 'Pause' button, 'Guide de découpe à gauche' and 'Guide de découpe à droite' (cut guides), a 'Début' (start) button, an 'Avertissement : fréquence de lecture faible' (warning: low reading frequency) icon, and a 'Réglage de la vitesse de lecture' (reading speed adjustment) slider. The bottom of the interface shows '105 of 249' and a warning icon.

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Copyright (c) 2015 Agfa HealthCare N.V., September 27, B-2040, Mortsel, Belgium.